THUASNE

SPORT

fr	Genouillère de maintien Thuasne Sport	3
en	Thuasne Sport knee support	4
de	Thuasne Sport-Kniebandage	5
nl	Thuasne Sport steunbrace voor de knie	7
it	 Ginocchiera di sostegno Thuasne Sport	8
es	Rodillera de sujeción Thuasne Sport	9
pt	 Joelheira de suporte Thuasne Sport	10
da	Knæstøttebind Thuasne Sport	12
fi	Thuasne Sport - polvituki	13
sv	Knästöd Thuasne Sport	14
el	Επιγονατίδα συγκράτησης Thuasne Sport	16
cs	Kolenní ortéza Thuasne Sport	17
pl	Podtrzymująca orteza kolanowa Thuasne Sport	18
lv	Ceļa atbalsts Thuasne Sport	20
lt	Palaikomasis kelio įtvaras „Thuasne Sport“	21
et	Thuasne Sport põlveortoos	22
sl	Opornica za koleno Thuasne Sport	23
sk	Kolenná ortéza Thuasne Sport	25
hu	Thuasne Sport támasztó térdmervítő	26
bg	Поддържаща наколеника Thuasne Sport	27
ro	Genunchieră de susținere Thuasne Sport	29
ru	Поддерживающий наколенник Thuasne Sport	30
hr	Thuasne Sport támasztó térdmervítő	31
zh	Thuasne Sport稳定护膝	33
ar	Thuasne Sport دعم الركبة	35

 cm	S 32 - 34	M 35 - 37	L 38 - 41	XL 42 - 45	XXL 46 - 49
--	---------------------	---------------------	---------------------	----------------------	-----------------------

fr	Circonférence de genou en légère flexion
en	Knee circumference in slight flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooid knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Ceļgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenкто kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
ru	Обхват колена в состоянии небольшого сгибания
hr	Obuhvaća koljeno u blagoj fleksiji
zh	轻度弯曲的膝关节周长
ar	محيط الركبة في حالة الثني الخفيف

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THUASNE SPORT KNEE SUPPORT

Description/purpose

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 5 sizes.

The device is composed of an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.

Composition

Polyamide - Elastan - Cotton.

Properties/Mode of action

Compressive elastic fabric ensuring proprioception.

Indications

Knee pain and/or swelling management.

Sport activities resumption.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace on the leg, making sure that the kneecap is properly positioned in the flexible knitted zone provided for this purpose.

Thuasne Logo must be on the upper part, centered and focused on the patella.

Care

Machine washable at 30°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2035001 (2022-08)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186